septodont

MOE 94 127 00 00

(fr)

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(en)

**EDTA Cream for Root Canal Cleaning and Smear Layer Removal**

Instruction For Use

Easier Filing with Lubricating Action

**APPLICATIONS**

Root canal preparation with files and reamers
Presence of calcification in the canal
Too narrow canals in posterior teeth
Smear layer removal

**MODE OF ACTION (MECHANISM PRINCIPLE)**
Removal of inorganic substances by chelation (EDTA - Ca)

Easier filling due its lubricating action
Produces the cleanest canal wall after smear layer removal

**COMPONENTS**

Canal + Cream 7,0 g (5.4 ml) x 2

- Disposable tips
- Net weight : 14 g

**DIRECTIONS FOR USE**

- Deposit a small amount of Canal + Cream in the canal orifice.

- Carefully pump into the canal with a fine file or reamer.
- Re-apply Canal + Cream when necessary or until cleaning and shaping is complete.

**HOW TO USE ROTARY RING**

- If in contact with air, content could be hardened, so it is necessary to cover a cap on the needle after use.
- Putting a syringe into a rotary ring is more convenient to handle & Use the rotary ring as a wrench to tighten or loosen a tip.

**CAUTIONS**

- Use Canal + Cream with discretion to prevent perforation.
- Avoid extrusion of the solution into the apex.
- Keep the syringe capped all the time.

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