COMPOSITION

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(EN)

COMPOSITION

Potassium nitrate, 2-hydroxyethyl methacrylate, Sodium fluoride, Hydroxyethylcellulose, Imidurea, Glycerol, Purified water, Artificial cherry flavor.

PROPERTIES

ISODAN is used for the treatment of dentinal hyperesthesia caused by any irritating agent on dentin: physical (cold, contact, heat), chemical (acid erosion), physiological (abrasion, tooth erosion).

INDICATIONS

Dentinal hyperesthesia.
Can be used on all exposed dentin surfaces, crown preparations and cervical erosion.

CONTRA INDICATIONS

Hypersensitivity to one of the components.

ADVERSE EVENTS

Allergic reaction.

INSTRUCTIONS FOR USE

- Isolate one or several teeth to be treated using a cotton roll,
- Carefully dry the surface of the area to be treated,
- Squirt some product into a Dappen dish,
- Take a small amount with an application brush,
- Apply a thin layer of product,
- Spread it carefully with an air jet,
- Reapply two or three times until the surface is shiny.

IN CASE OF PERSISTING SYMPTOMS, REAPPLY AT A ONE-WEEK INTERVAL.
Do not exceed 3 applications in the sequence of treatment.

WARNINGS AND PRECAUTIONS FOR USE

Warning
Contains 2-hydroxyethyl methacrylate
Causes skin irritation.
Causes serious eye irritation.
May cause an allergic skin reaction.
Refer to safety data sheet (available upon request).

STORAGE

Do not store above 25°C.

PRESENTATION

1 bottle containing 5 ml of solution + 20 application brushes.

For information:
The content of the bottle is estimated to ensure a maximum of 250 uses.

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