**COMPOSITION**

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**EN****COMPOSITION**

Chlorhexidine digluconate (20 % solution) ..... 0,2 g  
Denatured ethyl alcohol ..... 40,0 g  
Excipients q.s. ..... 100,0 g  
(acetic acid, purified water)

**PROPERTIES**

R4 is a solution for the cleaning of root canals by a final irrigation. The presence of chlorhexidine digluconate permits the bacterial disinfection canals, because of its specific action on anaerobic strains in the root-canals flora and endo-periodontal lesions.

**INDICATIONS**

Root-canal irrigation when endodontic re-treatment is required and/or when root-canal treatment requires several sessions.

**CONTRA INDICATION**

Hypersensitivity to any component.

**INSTRUCTIONS FOR USE**

After chemical and mechanical root-canal preparation, flush the canal with sterile water or an EDTA solution. Dry the canal with paper points. For the final irrigation, use R4 solution in a disposable pipette, preventing extrusion of the solution beyond the apical foramen. Do not flush. Remove the excess solution which could adversely affect

the sealing process of the temporary crown restoration. If necessary, place a cotton pellet in the access cavity and seal it hermetically with a temporary filling, ensuring that the cotton pellet is completely covered. Use a new pipette each time you take some solution.

At the next session (after 24 hours to 15 days), flush the canal with sterile water or an EDTA solution. Proceed with the chemical and mechanical preparation of the root-canal and/or fill the cavity with a permanent filling, depending on the clinical situation and according to routine procedures.

**WARNINGS AND PRECAUTIONS FOR USE**

When irrigation associates sodium hypochlorite and chlorhexidine, rinse with water between the two irrigants to avoid any interaction and precipitate formation.

**Danger.**  
Highly flammable liquid and vapour.  
Causes serious eye irritation.  
Refer to safety data sheet (available upon request).

**STORAGE**  
Close container tightly after each use.  
Use within 6 months after opening.  
Keep in a cool place (10 to 25°C).

**PRESENTATION**

13-ml bottle  
**For professional dental use only**

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