FR

**COMPOSITION**

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EN

**COMPOSITION**

Cord impregnated with an astringent solution containing aluminium and potassium sulfate.

**PROPERTIES**

Before taking impressions, preparing the gingiva is necessary. Therefore, to obtain precise models, it is essential to carry out a slight gingival retraction.

The use of RACESTYPTINE CORD N allows a temporary and immediate gingival retraction for impression taking and certain surgery procedures.

RACESTYPTINE CORD N is indicated when indirect impressions are taken.

The use of RACESTYPTINE CORD N, related to its haemostatic action, stops gingival bleeding and exudation of the mucosa.

**INDICATIONS**

Retraction of the gum before taking impressions.

Minor gingival haemorrhages.

**INSTRUCTIONS FOR USE**

To be used according to the usual technique of gingival retraction.

To remove necessary length of cord, lift dispensing lever. Pull required length of cord and cut, leaving a sufficient piece extruding to be able to grasp. Push lever down to block exit of the cord.

Insert the cord into the gingival sulcus and leave in place for about 5 minutes. Then, gently remove the cord making sure that no fragment remains onto the gingiva and rinse if necessary.

**PRESENTATIONS**

Bottle containing 2 metres of cord No 0 (thin)

Bottle containing 2 metres of cord No 1 (medium)

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