SEPTODONT petite

Seringue d'injection à aspiration / Aspirating Dental Injection Syringe
Aspirationssspritze für Dentalinjektionen / Dentale injectiespuit met aspiratie
Jeringa de inyección con aspiración

F

MODE D'EMPLOI

1. Insertion de la cartouche

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INSTRUCTIONS FOR USE

1. Insertion of dental cartridge

Gently retract handle of syringe until interior plunger cylinder moves to or below opening of cartridge window. Place dental cartridge into the syringe. Release handle to lock in cartridge.

2. Setting harpoon

Push forward on the plunger handle to set the harpoon. It is easier to set the harpoon before attaching a needle.

3. Attaching dental needle

Attach dental needle by threading onto the hub of the dental syringe. Continue threading until needle hub contacts syringe hub.

4. Aspiration of needle

Remove needle cap and release a small amount of solution to verify the needle is clear. To aspirate, gently pull back on the thumb ring.

Aspiration should be done after placement of needle in the mouth and during the injection.

5. Removing dental cartridge

Always cover needle before changing cartridges or for final disposal of needle and dental cartridge. To remove cartridge, pull syringe handle down until harpoon disengages from cartridge stopper. Then, turn syringe over and remove cartridge.

PRECAUTIONS FOR USE

• Warnings

Clean and sterilize the syringe before the first and after every use. Do not use detergents or disinfectants containing acid or alkaline additives.

• Proper maintenance and sterilization of instruments with aluminium components or anodized surfaces

After each use, remove all traces of blood and saliva by brushing and washing. Then rinse and dry the syringe. The most gentle method of mechanical cleaning consists of using completely demineralized water and a mild cleaning agent with a neutral pH-value.

Sterilize in autoclave between 134°C and 138°C (max 200°C) for the duration of the machine cycle (cycles may vary according to the regulations and/or the law in force in the country involved). When needed, lubricate the piston with instrument silicone oil.

After sterilization, store the syringe in a sterile and dry place.

Guarantee

Aspirating syringes are guaranteed against manufacturing or workmanship defects for 5 years.

Damages due to improper handling or care are not covered.

Certain dry-heat sterilizers and solutions found in chemical autoclaves and ultrasonics may tarnish color finish of syringe. Discoloration not covered by 5 years guarantee.

For professional use only in the practice of dentistry.

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4. Aspiration of needle

Remove needle cap and release a small amount of solution to verify the needle is clear. To aspirate, gently pull back on the thumb ring. Aspiration should be done after placement of needle in the mouth and during the injection.

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After each use, remove all traces of blood and saliva by brushing and washing. Then rinse and dry the syringe. The most gentle method of mechanical cleaning consists of using completely demineralized water and a mild cleaning agent with a neutral pH-value. Sterilize in autoclave between 134°C and 138°C (max 200°C) for the duration of the machine cycle (cycles may vary according to the regulations and/or the law in force in the country involved). When needed, lubricate the piston with instrument silicone oil. After sterilization, store the syringe in a sterile and dry place.

Guarantee

Aspirating syringes are guaranteed against manufacturing or workmanship defects for 5 years.
Damages due to improper handling or care are not covered.
Certain dry-heat sterilizers and solutions found in chemical autoclaves and ultrasonics may tarnish color finish of syringe. Discoloration not covered by 5 years guarantee.

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