## Ultra Safety Plus Twist PART B

## Sterilisable

**FR - Poignée stérilisable**  
**EN - Sterilisable handle**  
**ES - Émbolo esterilizable**  
**DE - Sterilisierbarer Griff**  
**IT - Impugnatura sterilizzabile**  
**PT - Cabo esterilizável**  
**NL - Steriliseerbare handgreep**  
**PL - Uchwyt przeczyszczony do ponownej sterylizacji**  
**EL - Αποστειρωθή λαβή**  
**BG - Стерилизируема дръжка**  
**CS - Sterilizovatelná rukojeť**  
**DA - Håndtag, der kan steriliseres**  
**ET - Steriliseeritav käepide**  
**HU - Sterilizálható fogantyú**  
**LT - Sterilizuojama rankena**  
**LV - Sterilizējams rokturis**  
**RO - Mâner sterilizabil**  
**SK - Sterilizovatelná rukoväť**  
**SL - Ročaj, ki se lahko sterilizira**  
**SV - Autoklaverbart handtag**

(FR)

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**DISPOSAL OF THE PRODUCT:**  
 The handle shall be disposed of in the clinical waste bags, or in any other container according to the regulations and/or the requirements specific to the country.

(EN)

**INTENDED USE**  
 Sterilisable handle to be used in combination with injection device for the injection of dental local anaesthetic.  
 Indications for use  
 Safe routine dental local anaesthetic injections.  
 Intended user  
 Dental Healthcare Professionals.  
 Clinical benefit  
 In combination with injection device, allow the administration of dental anaesthetics to patients who need to go through dental procedure requiring dental anaesthesia.

**PERFORMANCES**  
 Compliant with the O-Ring.  
 In combination with the injection device, allows the administration of dental local anaesthetic.  
 Technical Specification.  
 Blue sterilisable handle marked "TWIST"  
 For use with ULTRA SAFETY PLUS Twist Part A/ULTRA SAFETY PLUS Twist XL Part A injection devices.  
 Allows both active and passive aspiration.  
 Locking mechanism designed for assembly of the injection device and the handle.

**CONSTRUCTION FOR USE:**  
 For instruction on how to assemble the handle with the injection device refer to ULTRA SAFETY PLUS Twist Part A/ULTRA SAFETY PLUS Twist XL Part A.

**AVERTISSEMENT**  
 The handle is delivered non-sterile and must be sterilised before and after first use.  
 Do not re-sterilise more than 100 cycles. Otherwise, technical performances are not guaranteed.

**LIMITATIONS OF PROCESSAMENTO**  
 Maximum 100 cycles.

**TREATMENT INITIAL EN EL PUNTO DE USO**  
 Write the sterilisation notice as soon as possible after use.

**PREPARATION**  
 To disassemble the three parts of the handle:

1. Remove the O-ring.  
 2. Slide the plunger rod out of the finger-grip.

**CLEANING: AUTOMATED OR MANUAL**  
 Equipment required:  
 Washer-disinfectant compliant with a standard of the ISO 15883 series.

**MANUAL**  
 Material necessary:  
 Detergent: bath of limpezia enzimático (pH<10).  
 Brush.  
 Method:  
 Brush the three parts of the handle to remove excess sludge.  
 Soak all parts in a bath of detergent. Follow the instructions from the detergent manufacturer for the appropriate dilution and time for safe use.  
 Use a brush to apply detergent solution al all surfaces. Ensure that the full depth of the finger-grip holes is reached.  
 Rinse all parts with clear water.  
 Dry all parts.

**DISPOSAL OF THE PRODUCT:**  
 The handle shall be disposed of in the clinical waste bags, or in any other container according to the regulations and/or the requirements specific to the country.

**STERILISATION:**  
 Read the sterilisation instructions.  
 This handle is sterilisable up to 100 times.

**PRESENTATION:**  
 1 handle + 5 O-rings

**STERILISATION & REPROCESSING INSTRUCTIONS**  
**WARNINGS**  
 • Do not resterilise more than 100 cycles. Otherwise, technical performances are not guaranteed.

**LIMITATIONS ON PROCESSING**  
 Max 100 cycles.

**INITIAL TREATMENT AT THE POINT OF USE**  
 Write up necessary.

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**PREPARATION**  
 Dissassemble the three parts of the handle:  
 1. Remove the O-Ring.  
 2. Slide the plunger rod out of the finger-grip.

**CLEANING: AUTOMATED OR MANUAL**  
**AUTOMATED**  
 Equipment required:  
 Washer-disinfectant compliant with a standard of the ISO 15883 series.

**MANUAL**  
 Material necessary:  
 Detergent: enzymatic cleaner bath (pH<10).  
 Brush.  
 Method:  
 Brush the three parts of the handle to remove excess sludge.  
 Soak all parts in a bath of detergent. Follow the instructions from the detergent manufacturer for the appropriate dilution and time for safe use.  
 Use a brush to apply detergent solution al all surfaces. Ensure that the full depth of the finger-grip holes is reached.  
 Rinse all parts with clear water.  
 Dry all parts.

**DISPOSAL OF THE PRODUCT:**  
 The handle shall be disposed of in the clinical waste bags, or in any other container according to the regulations and/or the requirements specific to the country.

Phase	Temperature (°C)
Pre-Wash	cold
Detergent Wash	96
Rinse	60
Final rinse	93
Dry	116

**MANUAL**  
 Material required:  
 Detergent: enzymatic cleaner bath (pH<10).  
 Brush.  
 Method:  
 Brush the three parts of the handle to remove excess sludge.  
 Soak all parts in a bath of detergent. Follow the instructions from the detergent manufacturer for the appropriate dilution and time for safe use.  
 Use a brush to apply detergent solution al all surfaces. Ensure that the full depth of the finger-grip holes is reached.  
 Rinse all parts with clear water.  
 Dry all parts.

**DISPOSAL OF THE PRODUCT:**  
 The handle shall be disposed of in the clinical waste bags, or in any other container according to the regulations and/or the requirements specific to the country.

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