FR

**PRESENTATION**

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**INSTRUCTIONS FOR USE**

**Step 1:** Tear the pouch in the middle as shown.  
**Step 2:** Allow the injection device to fall onto the table, holding the packet in both hands (avoid handling the device so that the product's sterility is not altered).  
**Step 3:** Grip the barrel firmly and fully insert the Cartridge into the open end of the device.  
**Step 4:** Grip the handle plunger and put the thumb behind the finger holder. Introduce the handle tip into the barrel of the device, behind the cartridge.

**Step 5:** Now slide the sheath protecting the needle backwards, towards the handle until it CLICKS (the click is made as the sheath hits the handle and locks the unit together).  
**Warning:** Failure to retract the sheath fully until you hear the click may result in the device disassembling during use.  
**Step 6:** All movements now are away from the needle.  
Remove the needle cap and discard. The device is now ready to use. You should always ensure a minimum of 0.25 ml to 0.35 ml of solution has been previously expelled from the cartridge (providing space).  
**Self Aspiration:** At the base of the cartridge barrel of the injection device you will see there is a small protuberance. It appears as a plastic tip holding the centred back end of the needle that penetrates the diaphragm of the cartridge when inserted. At the beginning of the injection, the diaphragm presses against the protuberance, and aspiration occurs as the diaphragm moves back away from the protuberance when released.  
**Aspiration:** The second form of aspiration can be obtained by the pliable O-Ring that is situated at the end of the handle shaft (piston) this creates a vacuum when the handle is pulled back by the practitioner. The cartridge bung follows the piston providing active aspiration. It is best observed when a minimum of 0.25 ml to 0.35 ml of solution has been previously expelled from the cartridge (providing space).  
**Warning:** Should the device be inadvertently fully locked in position 2, no attempt whatsoever should be made to unlock it. Use a new injection device.  
**Step 7:** After you have finished the injection, remove your fingers from the O-Ring Handle and grip the finger holder, slide the protective sheath down the barrel of the syringe until you reach the first holding position, then grip the barrel of the syringe and pull the protective sheath down further to the final locking position.  
**Step 8:** If you wish to remove the Cartridge from the Syringe then, pull the Handle plunger back fully (A), and then peel off the Handle (B). Then pull the finger grip back up the plunger until it comes to the O-Ring Handle then (C) push the plunger back into the Cartridge and remove the Cartridge. Now the injection device can be disposed of safely.

**WARNINGS**

- Prior to injection, remove and dispose of needle cap. DO NOT RECAP NEEDLE.
- Do not bend, break, or otherwise stress needles as serious injuries to you and/or your patient can occur.
- Do not insert needle to hub during injections, as needles can break and become lodged in patient's tissue, potentially causing serious permanent injury. Do not use short needles (< 30mm) when the expected depth in soft tissue approaches the length of the needle.
- Avoid extreme pressure and excessive needle movement during injection as this may cause needles to break, which may result in serious injury to you and/or your patient.
- Be especially vigilant when repositioning a needle in a patient who appears to be apprehensive.
- ULTRA SAFETY PLUS XL: Avoid either too rapid of an injection and/or excessive pressure as that may lead to increased patient pain during injection.
- Do not try to force the protective sheath off the lock.
- Never tap instruments with hand into semi full sharps box.

**CAUTIONS**

a) Store in original packaging until ready to use.  
b) Read the Instructions For Use supplied with the handle.  
c) For single use only: This device is for use in a single patient. Any reuse exposes other patients and the user to critical risks, including infection and trauma since the needle is no longer sterile and is more likely to break.  
If multiple injections are required, replace the injection device for each cartridge in order to minimize the risk of needle point damage.

**1 patient / 1 cartridge / 1 injection device**

d) Check if the protective pouch is intact before use. If the protective pouch is not intact, the injection device is not sterile and must not be used.  
e) Ensure protective sheath is attached to the handle with an audible click confirming proper locked on position.  
f) ULTRA SAFETY PLUS XL: Because XL needles have a larger lumen (bore) than other needles, prior to injection, soft tissue should be stabilized to reduce the risk of needle deflection.  
g) Should the device be inadvertently fully locked in position 2, no attempt whatsoever should be made to unlock it. Use a new injection device.  
h) When finished and protective sheath is moved to Position 2 and locked by hearing an audible click, remove cartridge by using procedure described in step 8. Dispose of the injection device then into a sharps container. Discard the cartridge in a glass container. The handle is now ready for autoclaving. Please read autoclaving instructions.  
i) Always place locked injection device with contaminated needle into sharps container.  
j) Choose sharps container suitable for consumption, minimum size 6 litres or more.

**CLEANING AND STERILISATION OF THE HANDLE**

**Caution: Clean and sterilise the handle before the first and after every use.**

- Remove all traces of blood and saliva by brushing and washing the handle.
- Disinfectant solution with a (pH<10) may be used, in accordance with manufacturers instructions for diluting and use.
- Sterilise in autoclave between 134° C and 138° C, for the duration of the machine cycle. These cycles may vary according to the regulations and/or the law in force in the country involved.
- Store the Handle in a place free from any contamination of any type.
- End of life is normally determined by wear and damage due to use.

**For professional dental use only**

**AU Sponsor:** Specialites Septodont Pty Ltd  
Eclipse Tower Level 16, 60 Station St, Parramatta NSW 2150  
Australia

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